

Tactical Tourniquet



Product Description: Tactical Tourniquet

 This tourniquet is suitable for hiking, camping, or outdoor activities; a reliable emergency tool.

Instructions for Use:

1. **Apply** the tactical tourniquet to the bleeding site and pass the injured limb through the strap loop.
2. **Tighten** the belt and fasten it back on itself. Wrap the strap around the limb and rotate the **windlass (rod)** until the bleeding stops.
3. **Place** the rod inside the clip and lock it in place.
4. **Check** the wound status. If heavy bleeding continues, tighten the tourniquet further.
5. **Adhere** the strap over the rod inside the limb clip.
6. **Secure** the rod and the clip with the small strap and **record the application time** on the white time-stamp surface.

Important Notes:

1. The location and time of application must be clearly marked. The tourniquet is strictly for use on **limbs**; do not use on the head, neck, or torso.
2. **Time Stamp:** Note the application time to monitor the duration and prevent potential blood poisoning or tissue damage.
3. The use of tourniquets is **prohibited** for individuals with pre-existing arterial or venous diseases in the limbs.
4. This is for **emergency use only**. It should not be applied for an extended period. Seek professional medical assistance immediately.

5. Whenever possible, the tourniquet should be used by **professionals** or under professional guidance.

Specifications:

- **Color:** Black
- **Material:** Nylon
- **Weight:** 70g
- **Dimensions:** 16 x 5 x 4 cm (6.3 x 1.96 x 1.57 inches)
- **Unfolded Length:** Approx. 95 cm (37.4 inches)